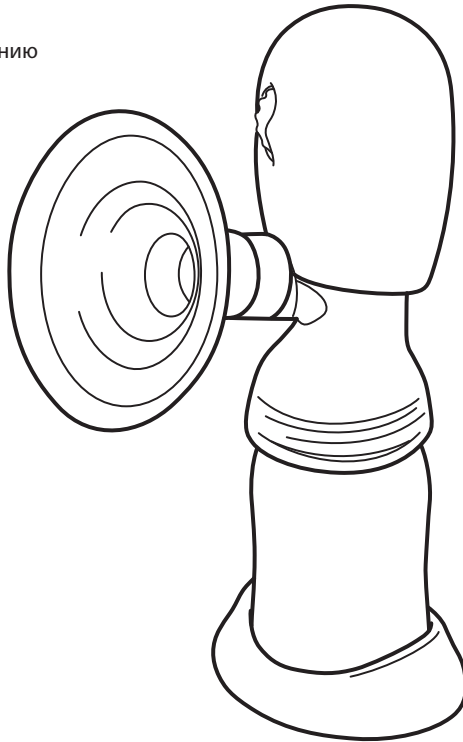
**EN** **Breast Pump**  
**Electric Portable**  
Instruction Manual

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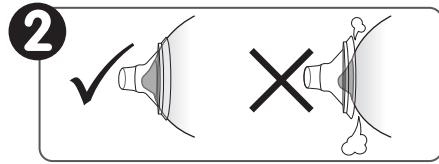
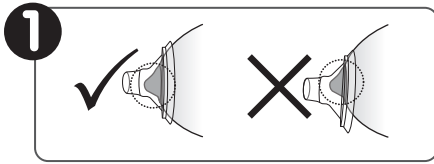
ID

TH

### EN Three things to remember when using the breast pump

To use the breast pump effectively, check the following:

- ❶ Make sure that your nipple/papilla is in the center of the flange.
- ❷ Ensure that there is a tight seal between your breast and the flange to prevent the occurrence of air leaks.
- ❸ Confirm that you can feel a suction sensation on your nipple/papilla. (The nipple/papilla/areola moves smoothly during suction.)

• **Breastmilk may not secrete as soon as you start expressing. Use a weak suction pressure to stimulate your nipple/papilla until ejection begins.**

Breastmilk ejection varies from woman to woman. Relax, and you will be an expert in no time.

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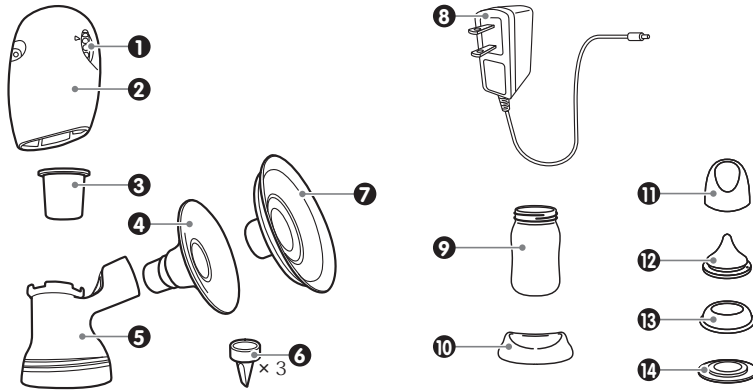
ALL

## EN

- 1 Dial 2 Motor 3 Soft cup 4 flange 5 Main body  
6 Silicone valve \* Two spare silicone valves are included. 7 Comfort ProperFit™ flange  
8 AC adapter 9 Bottle 10 Bottle stand 11 Hood 12 Nipple (SS size) 13 Cap 14 Sealing disk

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## Precautions for use

### Important Safeguards

When using electrical products, especially when children are present, basic safety precautions should always be maintained, including the following:

#### PLEASE READ THIS BEFORE USING THE PUMP

The following symbols indicate incorrect handling that can result in injuries or damage.

 <p><b>Warning</b></p> <p>Indicates incorrect handling that can cause major injury or death.</p>	 <p><b>Caution</b></p> <p>Indicates incorrect handling that can cause injury or property damage.</p>
---	---

The following symbols indicate important instructions that must be followed.

 <p><b>Instruction</b></p> <p>Indicates steps that must be taken.</p>	 <p><b>Prohibited</b></p> <p>Indicates actions that are prohibited.</p>
--	--

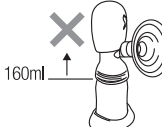
#### **Danger:** To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not use while bathing.
- Do not place or store this product where it can fall or be pulled into a tub or sink.
- Do not place or drop pump into water or other liquid.
- Do not reach for an electrical product that has fallen into water. Unplug immediately.

#### **Warning:** To reduce the risk of burns, electrocution, fire, or injury to persons:

- |   |   |
|---|---|
| <ul style="list-style-type: none"> <li> Close supervision is necessary when this product is used near children or invalids.</li> <li> This products should never be left unattended when plugged in.</li> <li> Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.</li> <li> Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.</li> <li> Wipe thoroughly, if there is dust on the AC adapter attachment plug. Failure to do so may cause fire.</li> <li> Do not disassemble or modify the AC adapter and motor. Doing so may cause fire as a result of electric shock or contamination.</li> <li> Do not insert or remove the AC adapter attachment plug with wet hands. Also do not handle batteries with wet hands. Doing so may cause electric shock.</li> </ul> | <ul style="list-style-type: none"> <li> Be careful not to damage the AC adapter. (Do not scratch, modify, place near a heater, bend excessively, twist, pull, place heavy objects on, or bundle it.) Doing so may cause fire, electric shock, or failure.</li> <li> Never use an AC adapter other than the one provided. Doing so may cause fire, electric shock, or product failure.</li> <li> Keep the cord away from heated surfaces.</li> <li> Never use while sleeping or drowsy.</li> <li> Never drop or insert any object into any opening or tubing.</li> <li> Do not use this product if it is broken or any parts are missing.</li> <li> Do not allow small children or pets to play with the pump unit, AC adapter, etc.</li> <li> Do not use outdoors with the AC adapter or operate where aerosol spray products are being used or where oxygen is being administered.</li> </ul> |
|---|---|

**! Caution**

- !** This breast pump is intended for use by a single user only.
- ⊘** Do not use excessively or for continued suction. Doing so may cause nipple pain.
- ⊘** Stop using the breast pump if your nipples are sore. Continued use may cause your symptoms to worsen.
- ⊘** Do not wipe with thinners, benzene, or alcohol. Doing so will cause deformities and damage.
- ⊘** Do not use detergents that contain abrasives. Doing so will cause scratches to the product.
- ⊘** Do not leave it near an open flame. Doing so will cause deformities and damage.
- ⊘** Do not expose the pump to severe shocks such as dropping or throwing. Doing so may cause malfunction or damage.
- ⊘** Do not express more than 160 ml of liquid into the bottle. The expressed breastmilk may overflow. 
- !** Prior to using the pump, check to make sure there are no abnormalities such as scratches, breaks, cracks, or deformities.
- !** Clean your hands before expressing.
- !** Wash and sterilize before using.


**Not working properly?**

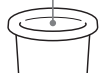**The motor is not working**

- Is it switched on?
- Is the AC adapter plugged in to the motor and outlet properly?
- Are the batteries inserted in the correct positive and negative direction?
- Are the batteries out of charge?

**There is no suction**

- Is the Comfort ProperFit™ flange firmly attached to the flange?
  - \* You may not feel sufficient suction if there is a gap between the flange and the Comfort ProperFit™ flange.
- Is the tip of the silicone valve torn?
  - \* There will be no suction if the silicone valve is torn or scratched. Replace it with the spare silicone valve.
- There may not be sufficient suction if the batteries are old. Replace the old batteries with new ones.
- Is each part assembled correctly?
  - Is the soft cup placed inside the main body?
  - Is the flange firmly inserted in the main body?
  - Is the soft cup turned inside out?
    - \* If you are unable to insert it properly, check the shape of the soft cup and try to insert it again.

	Direction of the soft cup	
	Correct direction ✓	Inside out ✗
View from diagonally above	No groove inside 	Groove inside 
Cross-sectional diagram		

## How to assemble

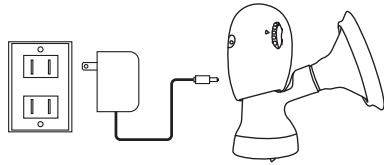
### Caution

- Prior to using for the first time, disassemble the pump and wash and sterilize the parts.
- Clean your hands before assembling.
- This pump can be disassembled into small parts. Do not let children put any parts in their mouth and do not lose any parts during assembly.
- Do not handle silicone rubber parts roughly. Doing so will cause tears or scratches.
- Always assemble the pump in the correct order.


### Using the pump with an AC adapter

- Insert the AC adapter plug firmly into the socket on the motor, and then insert the AC adapter into the outlet.
  - \* The AC adapter may become warm while in use. This is not a problem.
  - \* Always remove the adapter from the outlet when not in use.
  - \* Do not pull the adapter by the cord.

### Using the pump with batteries

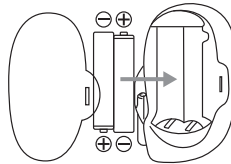- Remove the battery cover from the motor side.
- Insert two AA alkaline batteries in the correct + and - positions and replace the battery cover.
  - \* If you will not use the product for an extended period, remove the batteries.
  - \* Replace both batteries at the same time.
  - \* Do not use the product with a mix of new and old batteries.
  - \* Use alkaline batteries to ensure smooth suction pressure.


1


Hold the base of the silicone valve and attach it to the main body.

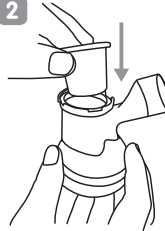2

Attach the soft cup to the main body.

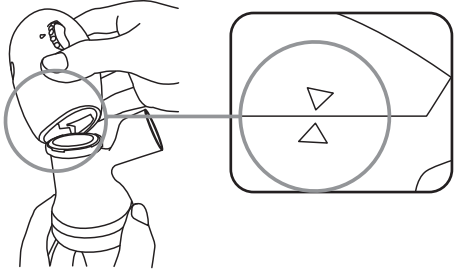3


Align the ▽ symbol on the motor with the △ symbol on the main body and then turn the motor to the right to lock it.

4


Turn the bottle to attach it to the main body.

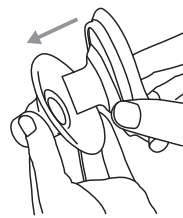5


Attach the Comfort ProperFit™ flange cover to the flange.

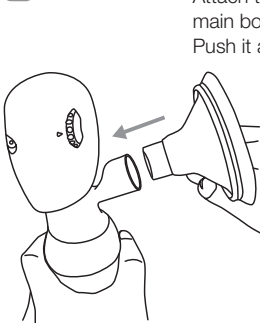6

Attach the flange to the main body. Push it all the way in.

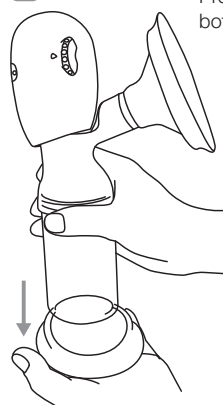7

Press the bottle into the bottle stand.

## How to use

### Caution

- Do not use a strong pressure at the start. Doing so may cause nipple pain.
- If you are unable to express any breastmilk, do not continue pumping. Doing so may cause nipple pain.
- Removing the flange from the breast while the switch is on may cause nipple pain. Always turn the switch off before removing the flange.
- Stop expressing if you are experiencing pain or discomfort.
- Take care not to knock over a bottle containing breastmilk because the contents may spill.

**1** Wash your hands before use.

**2** Wash and sterilize the breast pump first and then assemble it properly.

**3** Sit and relax in a chair to begin expressing.

**4** Rotate the dial all the way counter-clockwise to set it to the "OFF" position.

**5** Center the flange over your nipple/papilla and press it against your breast. Seal it securely so that no air can escape. You can create a good seal by using your other hand to support your breast from underneath.

\* If your nipple is unable to fit within the Comfort ProperFit™ flange without rubbing, you may need the larger size (sold separately).

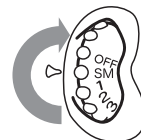
### **6** Stimulation mode

This mode gently stimulates your breast using a slow rhythm.

Stimulation mode starts when you rotate the dial to "SM"; one click clockwise from the "OFF" position.

Confirm that your nipple/papilla is moving and suction is being applied.

When breastmilk starts expressing (or after 1 to 2 minutes), rotate the dial to switch to Expression mode.

### **7** Expression mode

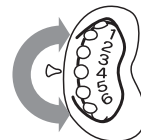
You can adjust the suction pressure using the six Expression mode levels, from Level 1 to Level 6.

(1: Gentle → 6: Strong)

Check each suction level before rotating to the next level.

Do not rotate the dial too quickly. Sudden strong suction may cause nipple pain.

It is important to express in comfort using the settings that feel right for you.

**8** Turn off the switch when you finish expressing.

**9** Remove the flange gently from your breast after you make sure the suction has stopped.

## How to take care

- Always disassemble the parts before washing and sterilizing. The motor and AC adapter cannot be sterilized.

### Motor

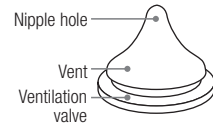
- Do not wipe the motor with a wet towel or wash with water. • If it is dirty, apply detergent to a cloth and wipe clean.

### Other parts

- Wash and sterilize the pump before using it for the first time. • Soak the pump in lukewarm water and wash them by using a detergent such as “Pigeon Liquid Cleanser” after use. Please make sure to rinse thoroughly after washing. • Sterilize the parts after washing. • Be careful not to lose small parts when washing the pump parts.
- Use a soft sponge brush for washing. • Do not handle silicone rubber parts roughly. Doing so will cause tears or scratches. • Do not clean with a scourer, polishing powder, or hard brush. Doing so will scratch the surface.

### Caring for the included nipple

- To protect the ventilation valve, a white powdered food additive has been applied. Before using for the first time, wash thoroughly.
- Use both hands to clean the ventilation valve, vent, and nipple hole by rubbing gently. If you scrub or pull on it too hard, the valve may split, and it causes leaks.
- The tip of the nipple is extremely delicate. Push the tip of the nipple with your finger when washing with a cleaning brush.

### [Boiling sterilization]

- Place thoroughly washed parts in a large pot filled with water. Bring to a boil. Sterilization time is 5 minutes after the water comes to a boil. • If there is not enough water, parts may stick to the side of the pot and become deformed. • Do not boil excessively. Doing so can quickly damage the product.

### [Steam sterilization]

- Use a sterilizer designed specifically for nursing bottles and nipples. Please read and follow the instructions that come with the product.

### [Chemical sterilization]

- Use a disinfectant designed specifically for nursing bottles and nipples. Please read and follow the instructions that come with the disinfectant. • The color of printing on the bottle may fade in case of chemical sterilization.

### [Microwave sterilization]

- Be sure to use containers that are designed specifically for microwave sterilizing. • Parts cannot be placed directly in the microwave for sterilization. \* Follow the sterilizing instructions that come with the product. \* Follow the instructions that come with your microwave.

## How to store expressed breastmilk

### Storage period

(The following are merely guidelines and do not provide any assurance of quality.)

	Refrigerator (4°C or below)	Freezer (around -18°C)
Freshly pumped breastmilk	24 hours	3 months (recommended) / 6 months (usable)
Thawed breastmilk (Thawed in refrigerator or under running water)	24 hours*	Do not refreeze.

\* If thawed by immersing in warm water, do not place back in the refrigerator.

### How to thaw frozen breastmilk

Thaw breastmilk using one of the following methods. • Immerse in warm water at around 40°C. • Place in the refrigerator. \*For up to 24 hours after taken out of the freezer • Place under running water.

### How to warm to a suitable temperature for breastfeeding

Immerse in warm water at around 40°C. • Breastmilk may separate during storage. This is normal and does not affect its nutritional content. Swirl gently before feeding. • When going out, it is recommended that you transport the breastmilk in a frozen state (without thawing it).

### Caution

- Do not thaw by immersing in boiling water, or by heating in the microwave or over an open flame. The nutritional content of the breastmilk may be damaged. • As it may not remain sanitary, do not let breastmilk thaw at room temperature. • Keep out of the reach of children.

## About the included nursing bottle and nipple

This nipple can only be used with Pigeon SofTouch™ Peristaltic PLUS™ Nursing Bottle.

Note: If your baby becomes comfortable with one certain nipple, he or she may reject new ones. So rotate 2 or more nipples and replace them about every 2 months. If the nipples become worn out, replace them without regard to how many times they have been used.

### For your child's safety and health

- Always use this product with adult supervision.
- Never use nipples as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Keep all components not in use out of the reach of children.
- Inspect before each use and pull the nipple in all directions. The nipples may be worn by biting. Special attention is needed for teething child. Throw away at the first signs of damage or weakness.
- Clean before each use.
- Do not leave a nipple in direct sunlight or heat, or leave in disinfectant ("sterilizing solution") for longer than recommended, as this may weaken the nipple.
- When not in use, keep in a dry, covered container.

### Warning

Follow the warnings below carefully. Not doing so can cause burns.

- The bottle may become hot when preparing formula. Be careful not to burn yourself.
- Do not prepare formula near your baby, as there is a danger of burns.
- Be sure to check the temperature of the milk or other fluid before feeding.
- The bottle may be hot after boiling or microwave sterilization. Be careful not to burn yourself.
- When preparing formula, if you shake the upright bottle, hot liquid may gush out, causing burns. Handle the bottle carefully.
- Do not attach the hood and shake the bottle. If the hood is attached, the high temperatures may cause the pressure inside the bottle to rise, leading to leaks or spouting of the hot liquid.

- Nursing bottles are for feeding your baby milk or other drink. Do not use them for any other purpose.
- The scale on the nursing bottle can be used as a guide when preparing formula. If you need to be more precise, use a measuring cup or measuring spoon.

### Formula

See the milk powder package for details on how to prepare formula.

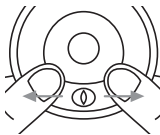
### Feeding

Feed your baby the formula after cooling it to approximately body temperature.

- Only a caregiver should feed the baby.
- If you tighten the cap too hard, the nipple may collapse.
- Consume the formula immediately once prepared. Discard any formula if you don't use within 2 hours of preparation.

### How to assemble nipple and cap

\*Insert the nipple into top of the cap.

Gently pull the ventilation valve on the nipple flange from behind and check whether the ventilation valve opens.

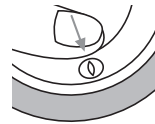
Grasp the shield firmly.

Place over the cap.

Press down firmly.

Press with your finger to make sure that the ventilation valve opens.

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EN

## Assurance

### Pigeon Portable Electric Breast Pump Warranty

**Thank you buying this Pigeon product; we hope you will be happy with it.**

- \* With this warranty, Pigeon guarantees the product to be free from defects in materials and workmanship for a period of 1(one) year from the date of purchase by its first use. Repairs or replacement under the warranty will be provided to you without charge.
- \* If still under the warranty period, prepare a proof of purchase documents, such as an invoice or receipt. And contact your nearest Pigeon subsidiary or distributor to make arrangements.
- \* The warranty does not apply to products used in the course of a business, trade or profession. The warranty does not cover any problems resulting from externally caused damages, use outside the product's specifications, faults caused by non-professional or non-approved repairs or from the use of non-Pigeon accessories and AC adapters.

**THE WARRANTY IS OFFERED AS AN ADDITIONAL BENEFIT TO YOUR STATUTORY RIGHTS AND DOES NOT AFFECT THESE RIGHTS IN ANY WAY.**

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EN

## Quality indication

Parts name	Material	Heat resistance	Sterilization Method
Main body / Flange / Bottle stand / Hood / Cap	Polypropylene	120°C	Boiling, Steam, Microwave, Chemical
Bottle	Polypropylene	110°C	
Comfort ProperFit™ flange / Soft cup / Silicone valve / Nipple / Sealing Disk	Silicone rubber	120°C	
Motor / AC adapter			

ALL

## Rated specifications

Motor	Power: DC 3.3 V (AC adapter or two AA alkaline batteries) Dimensions: 95 mm (height) x 62 mm (width) x 58 mm (depth) Weight: Approximately 160 g (without batteries)
AC adapter	Input: AC100-240v 50/60Hz Output: 3.3 VDC 1.0A 

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